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- Schmidt RE. Monoclonal antibodies for diagnosis of immunodeficiencies. *Blut*. 1989;59:200-206.
- Nicholson JKA. Use of flow cytometry in the evaluation and diagnosis of primary and secondary immunodeficiency diseases. *Arch Pathol Lab Med*. 1989;113:598-605.
- Foucar K, Goeken JA. Clinical application of immunologic techniques to the diagnosis of lymphoproliferative and immunodeficiency disorders. *Lab Med*. 1982;13:403-413.
- Cohen SB, Weetman AP. Activated interstitial and intraepithelial thyroid lymphocytes in autoimmune thyroid disease. *Acta Endocrinol*. 1988;119:161-166.
- Smolen JS, Chused TM, Leiserson WM, Reeves JP, Alling D, Steinberg AD. Heterogeneity of immunoregulatory T-cell subsets in systemic lupus erythematosus: correlation with clinical features. *Am J Med*. 1982;72:783-790.
- Giorgi JV, Hultin LE. Lymphocyte subset alterations and immunophenotyping by flow cytometry in HIV disease. *Clin Immunol Newsl*. 1990;10:55-61.
- Landay A, Ohlsson-Wilhelm B, Giorgi JV. Application of flow cytometry to the study of HIV infection. *AIDS*. 1990;4:479-497.
- Centers for Disease Control. 1997 Revised guidelines for performing CD4+ T-cell determinations in persons with human immunodeficiency virus (HIV). *MMWR*. 1997;46(No. RR-2):1-29.
- Nicholson JKA, Jones BM, Hubbard M. CD4 T-lymphocyte determinations on whole blood specimens using a single-tube three-color assay. *Cytometry*. 1993;14:685-689.
- Nicholson J, Kidd P, Mandy F, Livnat D, Kagan J. Three-color supplement to the NIAID DAIDS guideline for flow cytometric immunophenotyping. *Cytometry*. 1996;26:227-230.
- Nicholson JKA, Hubbard M, Jones BM. Use of CD45 fluorescence and side-scatter characteristics

- for gating lymphocytes when using the whole blood lysis procedure and flow cytometry. *Cytometry*. 1996;26:16-21.
- Antel J, Bania M, Noronha A, Neely S. Defective suppressor cell function mediated by T8+ cell lines from patients with progressive multiple sclerosis. *J Immunol*. 1986;137:3436-3439.
- Fitzgerald-Bocarsly P, Herberman R, Hercend T, et al. A definition of natural killer cells. In: Ades E, Lopez C, eds. *Natural Killer Cells and Host Defense*. Basel: Karger; 1989:1.
- Lanier LL, Le AM, Civin CI, Loken MR, Phillips JH. The relationship of CD16 (Leu-11) and Leu-19 (NKH-1) antigen expression on human peripheral blood NK cells and cytotoxic T lymphocytes. *J Immunol*. 1986;136:4480-4486.
- Haynes BF. Summary of T-cell studies performed during the Second International Workshop and Conference on Human Leukocyte Differentiation Antigens. In: Reinherz EL, Haynes BF, Nadler LM, Bernstein ID, eds. *Leukocyte Typing II: Human T Lymphocytes*. New York, NY: Springer-Verlag; 1986:3-30.
- Kan EAR, Wang CY, Wang LC, Evans RL. Noncovalently bonded subunits of 22 and 28 kd are rapidly internalized by T cells reacted with Anti-Leu-4 antibody. *J Immunol*. 1983;131:536-539.
- Knowles RW. Immunohistochemical analysis of the T-cell-specific antigens. In: Reinherz EL, Haynes BF, Nadler LM, Bernstein ID, eds. *Leukocyte Typing II: Human T Lymphocytes*. New York, NY: Springer-Verlag; 1986;1:259-288.
- Evans RL, Wall DW, Platsoucas CD, et al. Thymus-dependent membrane antigens in man: inhibition of cell-mediated lympholysis by monoclonal antibodies to the TH2 antigen. *Proc Natl Acad Sci USA*. 1981;78:544-548.
- Bernard A, Bousmell L, Hill C. Joint report of the first international workshop on human leukocyte differentiation antigens by the investigators of the participating laboratories: T2 protocol. In: Bernard A, Bousmell L, Dausset J, Milstein C, Schlossman SF, eds. *Leukocyte Typing*. New York, NY: Springer-Verlag; 1984:25-60.
- Cobbold SP, Hale G, Waldmann H. Non-lineage, LFA-1 family, and leucocyte common antigens: new and previously defined clusters. In: McMichael AJ, ed. *Leucocyte Typing III: White Cell Differentiation Antigens*. New York, NY: Oxford University Press; 1987:788-803.
- Wood GS, Warner NL, Warnke RA. Anti-Leu-3/T4 antibodies react with cells of monocyte/macrophage and Langerhans lineage. *J Immunol*. 1983;131:212-216.
- Perussia B, Acuto O, Terhorst C, et al. Human natural killer cells analyzed by B73.1, a monoclonal antibody blocking Fc receptor functions, II: studies of B73.1 antibody-antigen interaction on the lymphocyte membrane. *J Immunol*. 1983;130:2142-2148.
- Perussia B, Starr S, Abraham S, Fanning V, Trinchieri G. Human natural killer cells analyzed by B73.1, a monoclonal antibody blocking Fc receptor functions, I: characterization of the lymphocyte subset reactive with B73.1. *J Immunol*. 1983;130:2133-2141.
- Schmidt RE. Non-lineage/natural killer section report: new and previously defined clusters. In: Knapp W, Dorken B, Gilks WR, et al, eds. *Leucocyte Typing IV: White Cell Differentiation Antigens*. New York, NY: Oxford University Press; 1989:517-542.
- Ritz J, Trinchieri G, Lanier LL. NK-cell antigens: section report. In: Schlossman SF, Bousmell L, Gilks W, et al, eds. *Leucocyte Typing V: White Cell Differentiation Antigens*. New York, NY: Oxford University Press; 1995;2:1367-1372.
- Nadler LM. B Cell/Leukemia Panel Workshop: summary and comments. In: Reinherz EL, Haynes BF, Nadler LM, Bernstein ID, eds. *Leukocyte Typing II: Human B Lymphocytes*. New York, NY: Springer-Verlag; 1986;2:3-43.
- van Dongen JJM, Krissansen GW, Wolvers-Tettero ILM, et al. Cytoplasmic expression of the CD3 antigen as a diagnostic marker for immature T-cell malignancies. *Blood*. 1988;71:603-612.
- Brenner MB, McClean J, Dyalynas DP, et al. Identification of a putative second T cell receptor. *Nature*. 1986;322:145-149.
- Clevers H, Alarcon B, Wileman T, Terhorst C. The T cell receptor/CD3 complex: a dynamic protein ensemble. *Annu Rev Immunol*. 1988;6:629-662.
- Organizing Committee of the Fourth International Workshop on Human Leukocyte Differentiation Antigens. Appendix A: CD guide. In: Knapp W, Dorken B, Gilks WR, et al, eds. *Leucocyte Typing IV: White Cell Differentiation Antigens*. New York, NY: Oxford University Press Inc; 1989:1074-1093.
- Dalgleish AG, Beverley PCL, Clapham PR, Crawford DH, Greaves MF, Weiss RA. The CD4 (T4) antigen is an essential component of the receptor for the AIDS retrovirus. *Nature*. 1984;312:763-767.
- Maddon PJ, Dalgleish AG, McDougal JS, Clapham PR, Weiss RA, Axel R. The T4 gene encodes the AIDS virus receptor and is expressed in the immune system and the brain. *Cell*. 1986;47:333-348.
- Rudd CE, Burgess KE, Barber EK, Schlossman SF. Monoclonal antibodies to the CD4 and CD8 antigens precipitate variable amounts of CD4/CD8-associated p56lck activity. In: Knapp W, Dorken B, Gilks WR, et al, eds. *Leucocyte Typing IV: White Cell Differentiation Antigens*. New York, NY: Oxford University Press; 1989:326-327.
- Moebius U. Cluster report: CD8. In: Knapp W, Dorken B, Gilks WR, et al, eds. *Leucocyte Typing IV: White Cell Differentiation Antigens*. New York, NY: Oxford University Press; 1989:342-343.
- Gallagher PF, Fazekas de St. Groth B, Miller JFAP. CD4 and CD8 molecules can physically associate with the same T-cell receptor. *Proc Natl Acad Sci, USA*. 1989;86:10044-10048.
- Anderson P, Blue M-L, Morimoto C, Schlossman SF. Cross-linking of T3 (CD3) with T4 (CD4) enhances the proliferation of resting T lymphocytes. *J Immunol*. 1987;139:678-682.
- Eichmann K, Jonsson J-I, Falk I, Emmerich F. Effective activation of resting mouse T lymphocytes by cross-linking submitogenic concentrations of the T cell antigen receptor with either Lyt-2 or L3T4. *Eur J Immunol*. 1987;17:643-650.
- Perussia B, Trinchieri G, Jackson A, et al. The Fc receptor for IgG on human natural killer cells: phenotypic, functional, and comparative studies with monoclonal antibodies. *J Immunol*. 1984;133:180-189.
- Lanier LL, Chang C, Azuma M, Ruitenberg JJ, Hemperly JJ, Phillips JH. Molecular and functional analysis of human natural killer cell-associated neural cell adhesion molecule (NCAM/CD56). *J Immunol*. 1991;146:4421-4426.
- Schubert J, Lanier LL, Schmidt RE. Cluster report: CD56. In: Knapp W, Dorken B, Gilks WR, et al, eds. *Leucocyte Typing IV: White Cell Differentiation Antigens*. New York, NY: Oxford University Press; 1989:699-702.
- Cunningham BA, Hemperly JJ, Murray BA, Prediger EA, Brackenbury R, Edelman GM. Neural cell adhesion molecule: structure, immunoglobulin-like domains, cell surface modulation, and alternative RNA splicing. *Science*. 1987;236:799-806.
- Dorken B, Moller P, Pizzutto A, Schwartz-Albiez R, Moldenhauer G. B-cell antigens: CD19. In:

- Knapp W, Dorken B, Gilks WR, et al, eds. Leucocyte Typing IV: White Cell Differentiation Antigens. New York, NY: Oxford University Press; 1989:34-36.
43. Schwitzer R. Cluster report: CD45/CD45R. In: Knapp W, Dorken B, Gilks WR, et al, eds. Leucocyte Typing IV: White Cell Differentiation Antigens. New York, NY: Oxford University Press; 1989:628-634.
  44. Centers for Disease Control. Update: universal precautions for prevention of transmission of human immunodeficiency virus, hepatitis B virus, and other bloodborne pathogens in healthcare settings. MMWR. 1988;37:377-388.
  45. Protection of Laboratory Workers from Infectious Disease Transmitted by Blood, Body Fluids, and Tissue: Tentative Guideline. Villanova, PA: National Committee for Clinical Laboratory Standards; 1991. NCCLS document M29-T2.
  46. Nicholson JK, Browning SW, Orloff SL, McDougal JS. Inactivation of HIV-infected H9 cells in whole blood preparations by lysing/fixing reagents used in flow cytometry. J Immunol Methods. 1993;160:215-218.
  47. Procedures for the Collection of Diagnostic Blood Specimens by Venipuncture: Approved Standard. Villanova, PA: National Committee for Clinical Laboratory Standards; 1991. NCCLS document H3-A3.
  48. Clinical Applications of Flow Cytometry: Quality Assurance and Immunophenotyping of Peripheral Blood Lymphocytes; Tentative Guideline. Villanova, PA: National Committee for Clinical Laboratory Standards; 1992. NCCLS document H42-T.
  49. Giorgi JV. Lymphocyte subset measurements: significance in clinical medicine. In: Rose NR, Friedman H, Fahey JL, eds. Manual of Clinical Laboratory Immunology. 3rd ed. Washington, DC: American Society for Microbiology; 1986:236-246.
  50. Jackson AL, Warner NL. Preparation, staining, and analysis by flow cytometry of peripheral blood leukocytes. In: Rose NR, Friedman H, Fahey JL, eds. Manual of Clinical Laboratory Immunology. 3rd ed. Washington, DC: American Society for Microbiology; 1986:226-235.
  51. Prince HE, Hirji K, Waldbeser LS, Plaeger-Marshall S, Kleinman S, Lanier LL. Influence of racial background on the distribution of T-cell subsets and Leu 11-positive lymphocytes in healthy blood donors. Diagn Immunol. 1985;3(1):33-37.
  52. Angadi CV. Lack of Leu-3a epitope on T-helper (CD4) lymphocytes. J Clin Lab Anal. 1990;4:193-195.
  53. How to Define and Determine Reference Intervals in the Clinical Laboratory: Approved Guideline. Villanova, PA: National Committee for Clinical Laboratory Standards; 1992. NCCLS document C28-A.
  54. Lanier LL, Le AM, Phillips JH, Warner NL, Babcock GF. Subpopulations of human natural killer cells defined by expression of the Leu-7 (HNK-1) and Leu-11 (NK-15) antigens. J Immunol. 1983;131:1789-1796.

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